


**SIS (+)electrode**

Onto surrounding normal tissue as close as possible, no more than 2cm (3/4") from edge of wound; place electrode carefully not to disturb wound.

**SIS 'return' (-)electrode**

Onto intact skin as much as possible directly behind the centre of the wound on the opposite anatomical surface of injured body part.

**SUPERFICIAL WOUND (INFECTED)**

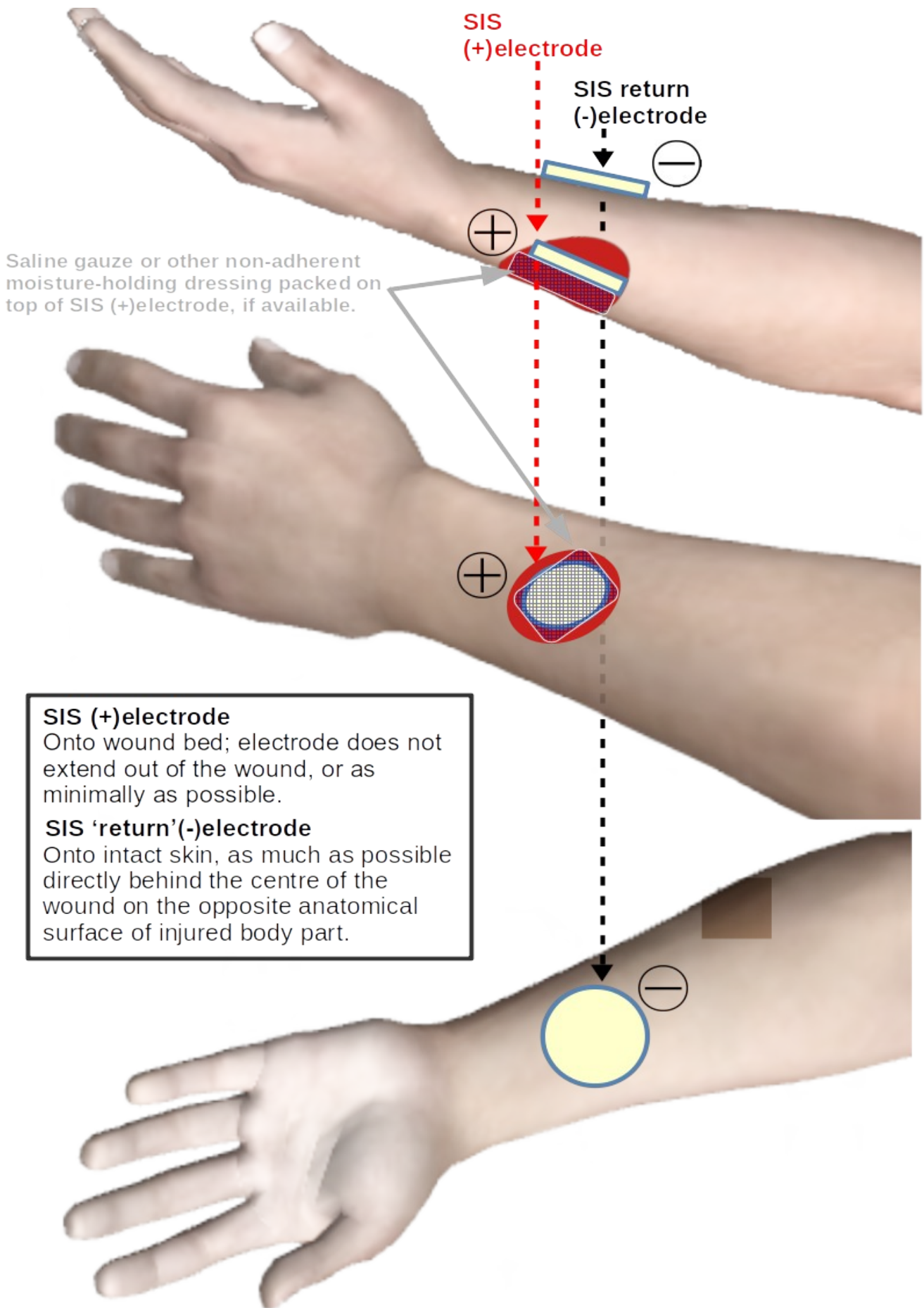


**DEEPER WOUND (INFECTED)**

**SIS Positive Electrode**

silver-nylon surface positioned directly over wound *and extending at least 2cm (3/4") beyond the edges of the wound on all sides.*

**SIS Return Electrode**

silver-nylon surface positioned onto normal skin as much as possible directly behind the wound on the anatomically opposite surface of the injured body part.

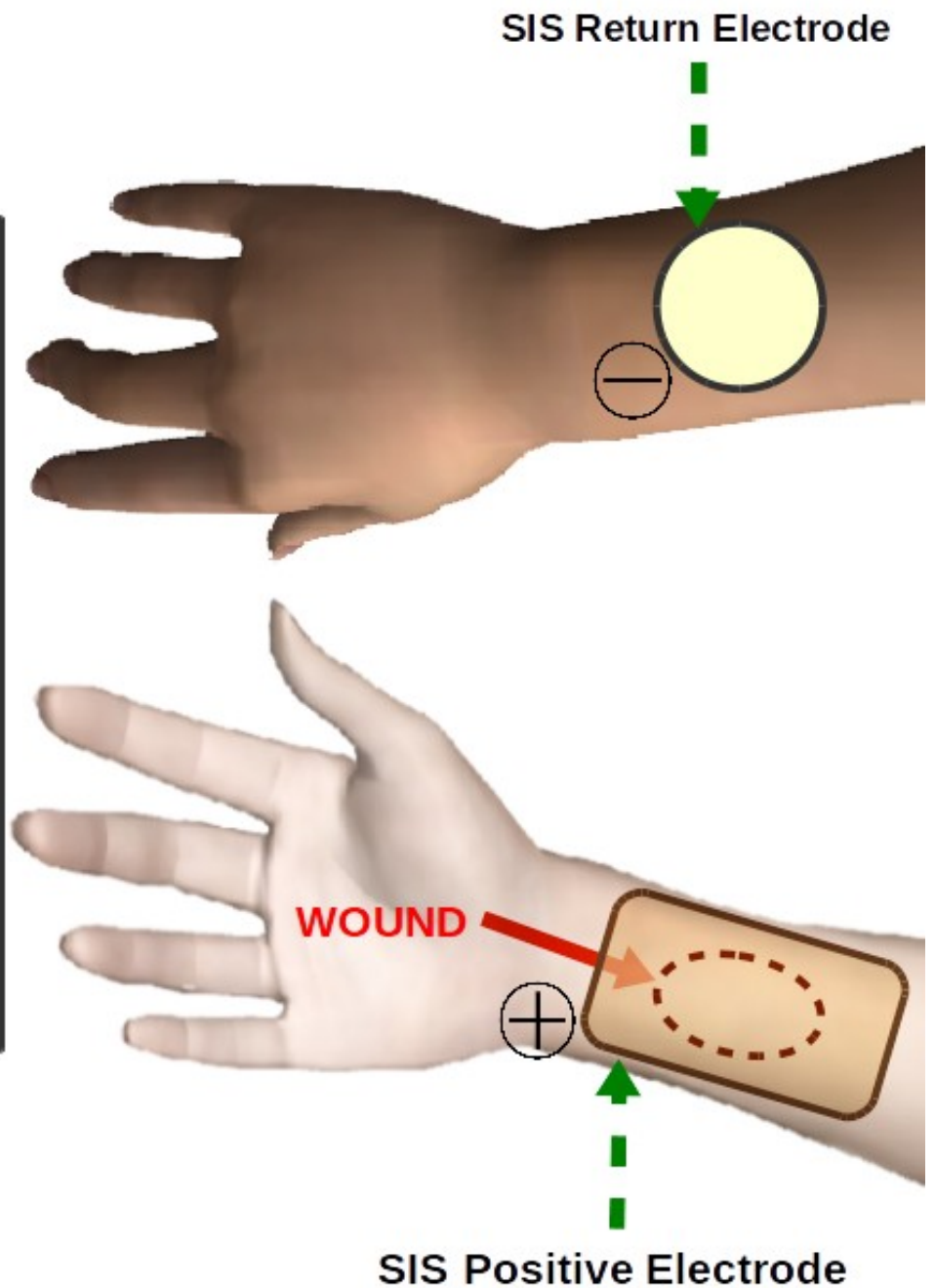

**FIRST AID: SIS (+) POSITIVE ELECTRODE ONLY,  
OR SIS (+) POSITIVE AND (-) RETURN  
ELECTRODES AND SIS MACHINE**