


## SUPERFICIAL WOUND (INFECTED)

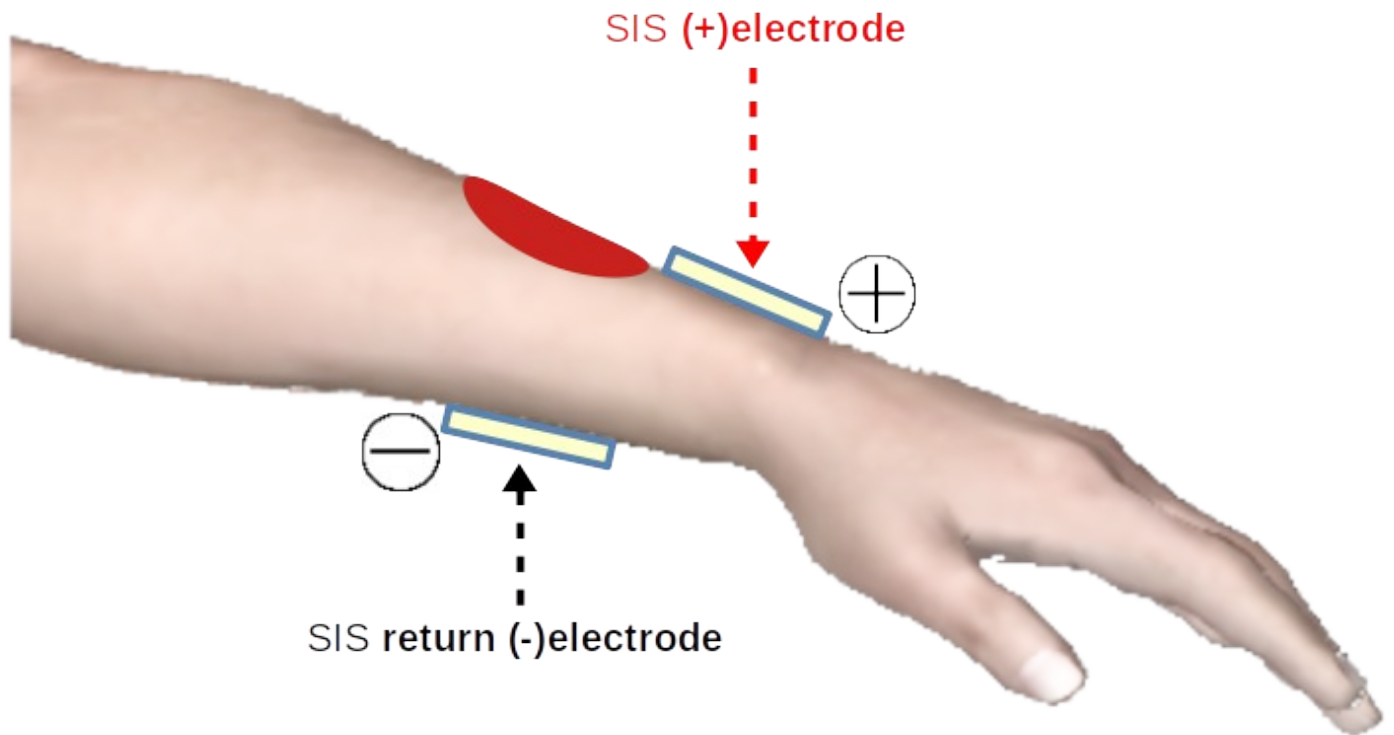


### **SIS (+)electrode**

On intact skin as close as possible and no more than 2cm (3/4") from edge of wound; electrode placed carefully not to disturb wound.

### **SIS return (-)electrode**


On intact skin as much as possible directly behind the centre of the wound on the opposite anatomical surface of injured body part.

# SUPERFICIAL WOUND (INFECTED) Across Wound Positioning

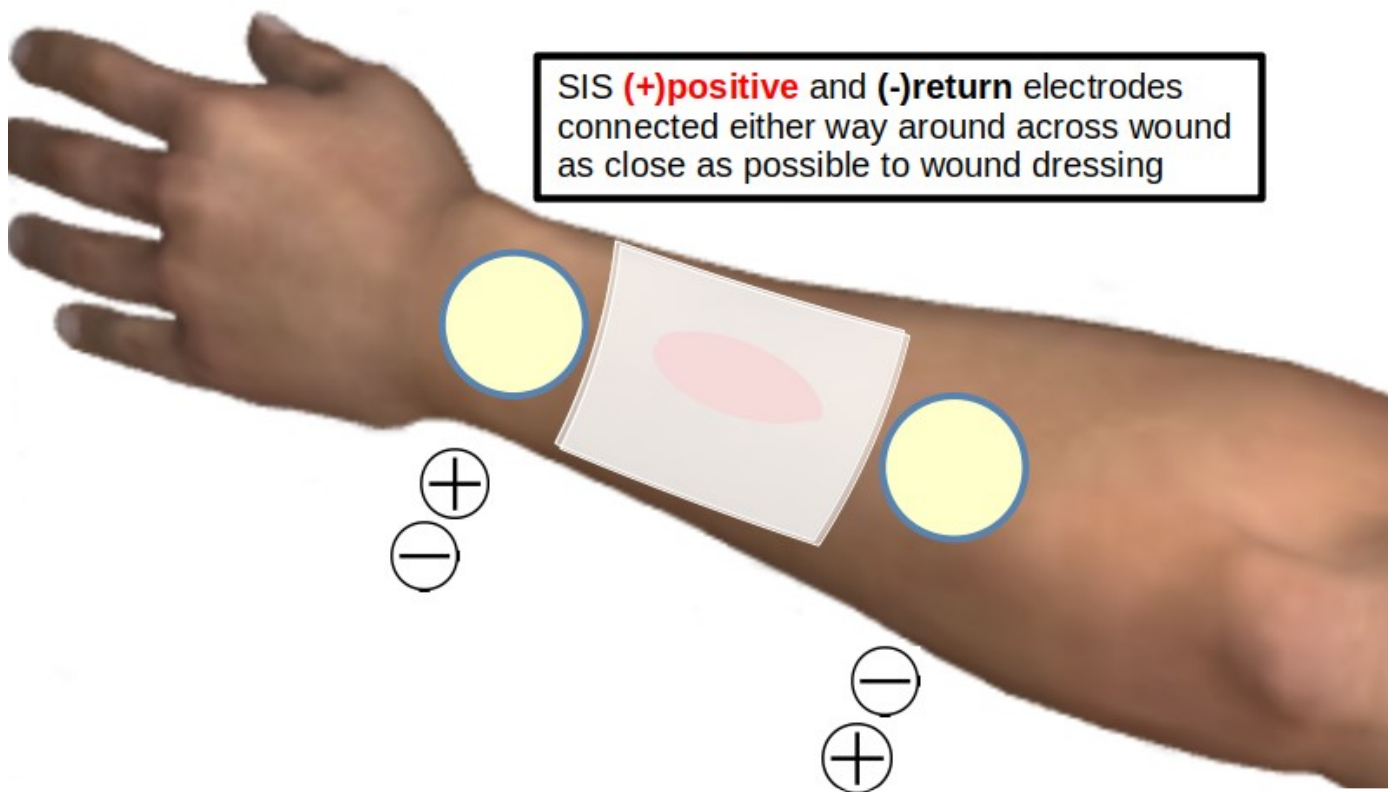

# DEEPER WOUND (INFECTED)

## Woundcare professional only

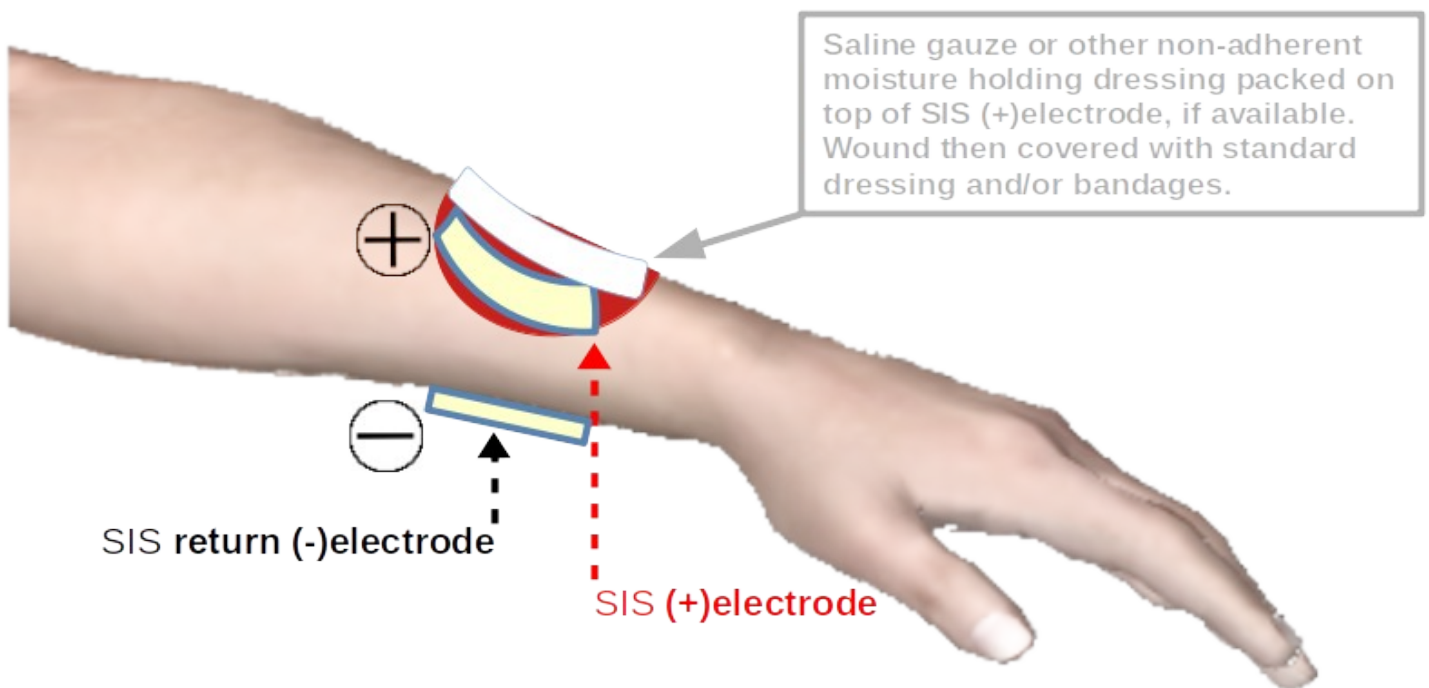


### SIS (+)electrode

On wound bed; electrode does not extend out of wound or as minimally as possible.

### SIS return(-)electrode

On intact skin as much as possible directly behind the centre of the wound on the opposite anatomical surface of injured body part.

# FIRST AID: SIS (+) POSITIVE ELECTRODE ONLY OR SIS (+) POSITIVE AND (-) RETURN ELECTRODES AND SIS MACHINE

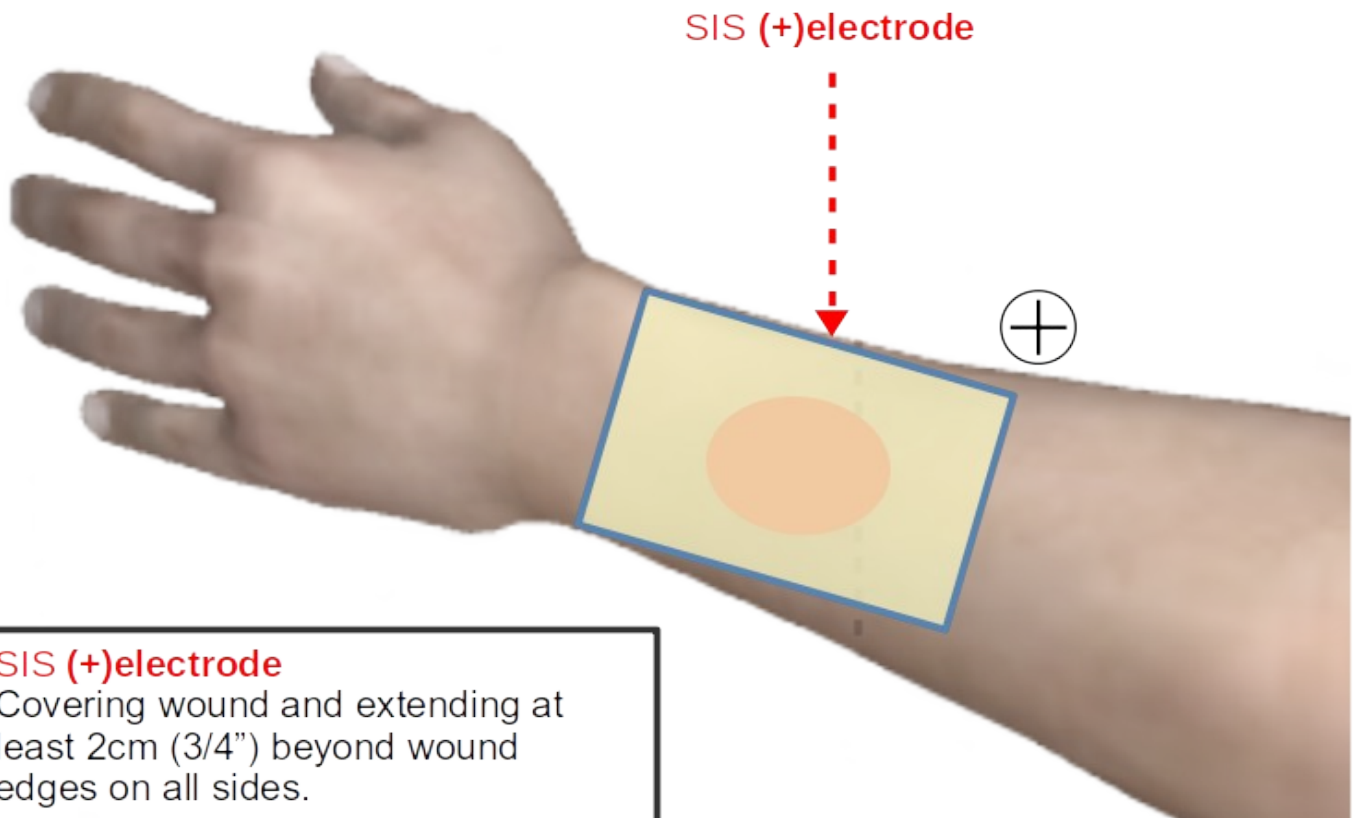

## SIS (+) electrode

Covering wound and extending at least 2cm (3/4") beyond wound edges on all sides.

**SIS return (-) electrode** if used  
On intact skin as much as possible directly behind the centre of the wound on the opposite anatomical surface of injured body part.

